

## OVERBED TABLE - ASSEMBLY INSTRUCTIONS



### CAUTION

DO NOT disassemble or use this Overbed Table without first reading and understanding the instruction sheet. If you are unable to understand the Warning or Instructions, contact your distributor before attempting to use or installing the Overbed Table. Otherwise, injury or damage may occur.

DO NOT overload the Overbed Table beyond its Safe Work Load capacity (15kg evenly distributed).

DO NOT lean on Overbed Table or use to support whilst standing or moving about.

DO NOT use if Overbed Table is damaged or components are missing. Before use, ensure all attaching hardware is securely tightened.

Ensure the table top is not touching any walls when adjusting the height. Keep hands and fingers clear...

DO regularly check frame screws and castors to ensure they are in good working order.

## USE & LIABILITY

The Overbed Table height is adjustable so that it can be positioned at the most ergonomically suitable height. Any other use is at user's risk. Under no circumstances does the manufacturer accept warranty claims or liability claims for damages caused from improper use or handling of the Overbed Table.

## WARRANTY

PremiumLift Overbed Table warranty period is strictly twelve (12). The warranty period commences on the day of actual delivery to the primary purchaser as reflected by your authorised reseller or distributor invoice provided to you.

The provision of this warranty shall not apply to any product that has been subject to abuse, misuse, neglect or accident and does not extend to normal wear and tear. Nor does the warranty apply to any product that has within the warranty period been used for a purpose for which the product has not been designed or which has been repaired or altered in such a way as to adversely affect its performance and reliability. Service by an unauthorised source may void the warranty.

If the Overbed Table does not function properly or requires repairs please contact your distributor.

# ASSEMBLING

Step 1 - Turn the table top upside down and lay on a flat surface, ensure you are clear of any hazardous materials such as liquid, electrical cables and heaters;

Step 2 - Secure column on to table top support with supplied screws;

Step 3 - Secure base on to column;

Step 4 - Double check that all screws and parts are tightly fixed onto the structure and place the Overbed Table in an upright position on a flat surface.

Step 5 - Remove column safety screw

