

# INSTALLATION

## 8 REMOVE PACKAGING

Remove cardboard cover from the bed. Please do not dispose of the cover. It can be used again as a dust cover in the event the care bed is later stored in the transport rack.

Do not remove the cardboard protection from the top of hand rail of both headboard or footboard sections at this time as it will protect the bed during assembly.



## 9 REMOVAL FROM THE TRANSPORT KIT

Lift the foot end of the bed platform from the transport kit. To prevent fingers being trapped, please hold the inside of the platform section and not on the outer frame.

Lay the platform section face down on the floor.

To remove the backrest section from the transport frame, lift and slide sideways between the head and footboard sections.



Remove the height-adjustable head and foot end panels from the transport frame by releasing the 4 lock fixings that are later used to fix the height-adjustable head and foot end panels to the mattress base. Lay the head and foot end panels (with the headboards facing upwards) at each end of the mattress platform.



# INSTALLATION


## 10 ASSEMBLING THE BED COMPONENTS

Lay the platform section face down on the floor adjacent to the other half. Slide the connecting bars into the other frame.



Once the platform sections are together install the final fixing screws.

Use the supplied allen key to tighten all 8 fixings (4 fixings on each side of the platform).

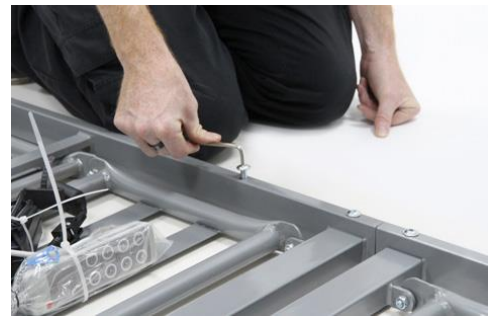

Place head end and foot hand on the floor.



# INSTALLATION

## 11 CABLES AND CONNECTORS

First, remove the plug cover by unscrewing the two fixing screws.

Connect the height adjustment motors and the knee break adjustment motor to the control box.

The backrest motor is supplied already plugged in. After inserting all plugs, screw the plug cover back onto the power supply unit housing.

Cut the cable ties securing the handset and the power cable to the bed chassis taking care not to damage the cables.

Fix the power cable to the chassis using the bracket at the head end of the bed, pull the cable taut and lock the cable grip. Secure the handset and power cable to the chassis using the cable clips attached to the platform.

Carefully turn over the sleeping platform, to not damage the raise/lower actuator cables

Insert both platforms to lift system connectors into the end of the platform. Install but do not tighten the 4 fixings as this will make connecting the platform to the lift systems easier.



# INSTALLATION

## 12 INSTALLING PLATFORM TO HEAD AND FOOT END

Connect the platform to the two lift systems, taking care not to trap the raise and lower actuator cables. Once connected lock all 4 pins on the platform to lift system connectors.

Do not use the bed unless the lock fixings are in place.

On each height-adjustable head and foot end, there is a cable guide roller, through which the cable of the raise/lower actuator should run.

Install the black cable guidance roller.

Cut the last cable ties holding the backrest and foot rest sections in place, remove the final cardboard protection from the head and foot sections but DO NOT cut the cable tie securing the backrest actuator.

Ensure the power supply is never positioned on or under the bed.

Avoid rolling the bed over the mains cable.

Insert the mains plug into the socket.

The mains plug must always remain accessible to enable immediate removal from the wall socket in case of emergency.

The electrical adjustment motors are now ready for use.

Remove all packaging.

Disassembly of the care bed is carried out in reverse order of assembly.

