QUICK INSTALLATION MANUAL: Deluxe Floor Bed

REMOTE GUIDE POSITIONS Remote Pair Memory A/B/C -Zero Gravity Backrest & Lock/Unlock Legrest Up Legrest Up Backrest Up-Flat Position Backrest Down Legrest Down Backrest & Legrest Down Hidden Castors Floorline Position Hi/Lo Hi/Lo Raise Lower Trendelemberg Seating Position Under Bed Light Premium Lift

Premium Lift



CAUTIONS

QR Code

Scan QR Code, download and read complete User Manual prior to using Deluxe Floor Bed "The Bed"

Do not sit on the head and leg section of the bed when operating the raise function.

Ensure the recommended service and maintenance schedule is completed. Failure to do so may invalidate warranty claims.

WARNINGS

Before lowering the bed check the area underneath is clear. Any obstructing items could be damaged or could cause instability to the bed.

Take care when lowering the bed to ensure limbs of any person or other objects present do not become trapped under the bed. Do not store the power supply under the bed.

INSTALLATION

- Lay The Bed on the floor, remove all components from carton and get rid of all packaging.
- Make sure motors and power supply are connected to the control box.
- Insert the mains plug into the socket
- Pair Handset

PAIRING REMOTE TO CONTROL BOX

- Locate PAIR button on Remote and Control Box
- 2. Press Remote Control PAIR button until blue light is flashing
- 3. Press Remote PAIR button twice

OKIN Remote RF2517

www.premiumlift.com.au