




QR Code

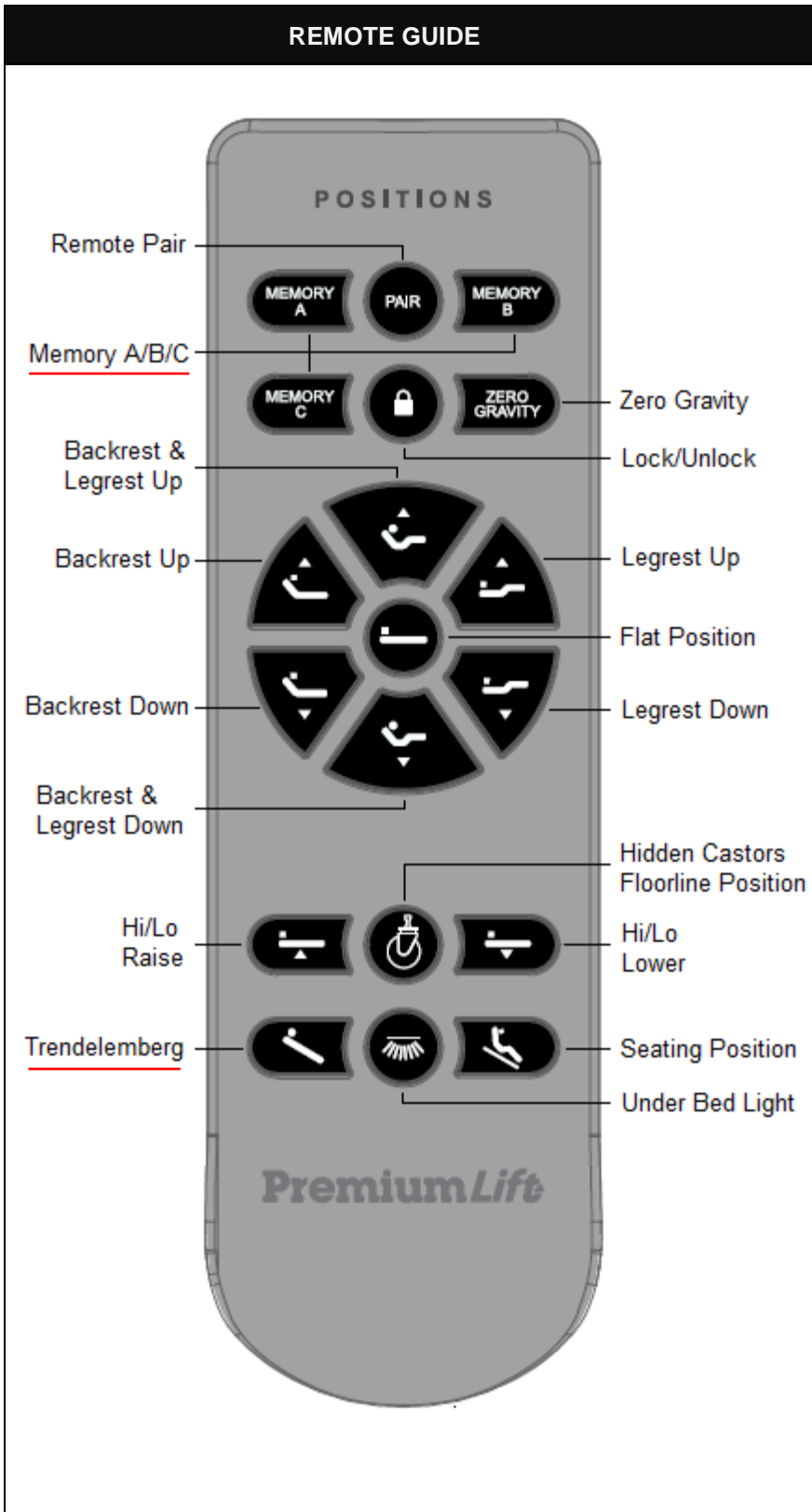

CAUTIONS

Scan QR Code, download and read complete User Manual prior to using Deluxe Floor Bed “The Bed”

Do not sit on the head and leg section of the bed when using Memory function

Ensure the recommended service and maintenance schedule is completed. Failure to do so may invalidate warranty claims.

WARNINGS

Before using Memory function check the area underneath is clear. Any obstructing items could be damaged or could cause instability to the bed.

Take care when lowering the bed to ensure limbs of any person or other objects present do not become trapped under the bed. Do not store the power supply under the bed.

Memory Buttons will not work if the bed is below Flat Position

Press Trendelemborg Button prior memorizing position

Memory function will become a pre-set position. To stock auto operation press any button

HOW TO USE MEMORY BUTTON?

- Adjust bed in desire position, press Trendelemborg Button for 1 second, now press and hold Memory Button for 10 sec. until all handset buttons backlight is flashing

OKIN Remote RF2517

www.premiumlift.com.au